




USER MANUAL

Please read this instruction manual thoroughly before use and keep for future reference.

MELBA RECLINER


Lift Chair with Heating Function

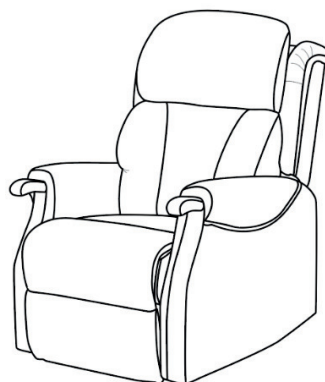

COMFORT CARE MELBA RECLINER

CONTENTS

1. **INTRODUCTION:** USER MANUAL
2. **SAFETY PRECAUTIONS:** WARNING
3. **INSTALLATION:** INSTALLATION OF CHAIR
4. **ASSEMBLY:** ASSEMBLY OF CHAIR
5. **SET UP:** ELECTRICAL CONNECTIONS
6. **SET UP:** BATTERY BACK UP
7. **USAGE:** CHAIR POSITION HANDSET CONTROL FUNCTIONS
8. **CARE:** PRODUCT CARE
9. **SPECIFICATIONS:** SPECIFICATIONS

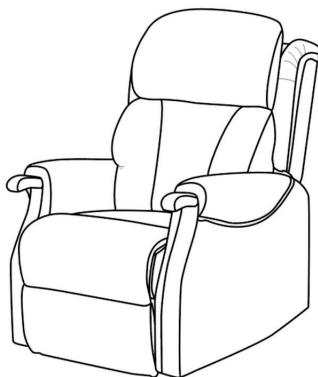


COMFORT CARE MELBA RECLINER

INTRODUCTION

This Lift Chair is designed to help people rise easily from the chair and to relax both body and mind with a heating function

Please read this instruction manual thoroughly before use and keep for future reference.

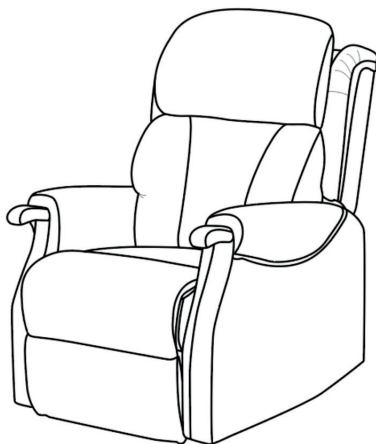

COMFORT CARE MELBA RECLINER

SAFETY PRECAUTIONS


WARNING

- The chair must be assembled and used according to the instructions in this manual to avoid damage and injury.
- Do not use in a room where the temperature is higher than 40°C.
- Do not use outdoors.
- Do not place heavy objects on the armrest, footrest or back of the chair.
- Do not jump on the chair.
- Children must be supervised by an adult when using the chair
- Check that the voltage corresponds with the specifications indicated for the chair.
- Do not unplug from the electrical outlet by pulling on the cord.
- Do not connect or disconnect the plug from the electrical outlet with wet hands.
- Do not place the adaptor cord under the chair.
- Do not let the adaptor cord hang down from the chair.
- Do not use the chair if the cord or plug is damaged.
- Do not operate the remote control with wet hands.
- Do not put your fingers and hands in between the seat and the footrest when the chair is reclining or when the chair is returning from the recline position to avoid injury.
- Do not use if water has been spilled on the chair, the upholstery is torn to expose the inside mechanism or you detect a malfunction or another abnormal condition during operation.
- Avoid using the chair during severe lightning storms.
- In the event of accidental damage or malfunction, switch off the power and disconnect the power cord from the electrical outlet.

COMFORT CARE MELBA RECLINER

INSTALLATION OF CHAIR

- **710mm** from wall
- **660mm+** in front

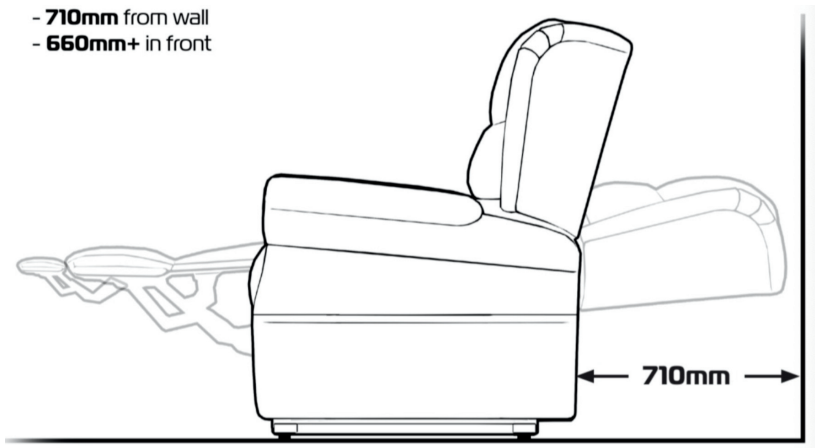


- Allow sufficient space in front of and behind the chair for reclining.
- To allow for unimpeded reclining, make sure there is a minimum of 65 cm behind the chair.
- Ensure there is at least 40cm in front of the chair.
- Place on a flat, level floor.
- The area between the seat and the footrest should be kept.

COMFORT CARE MELBA RECLINER

ASSEMBLY OF CHAIR

The chair is packed in two pieces consisting of the back and the base to make transport and delivery easier.


Check all parts carefully before assembling the chair.

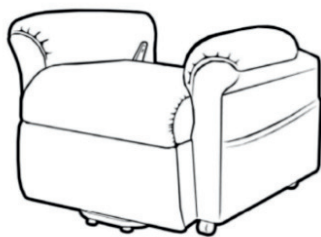

1x Seat



2x Wing

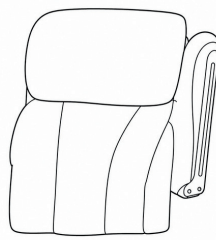


1x Back

The back of the chair has a receiver bracket on each side.

The rear of the base of the chair has a vertical flat bar on each side.

COMFORT CARE MELBA RECLINER

ASSEMBLY OF CHAIR

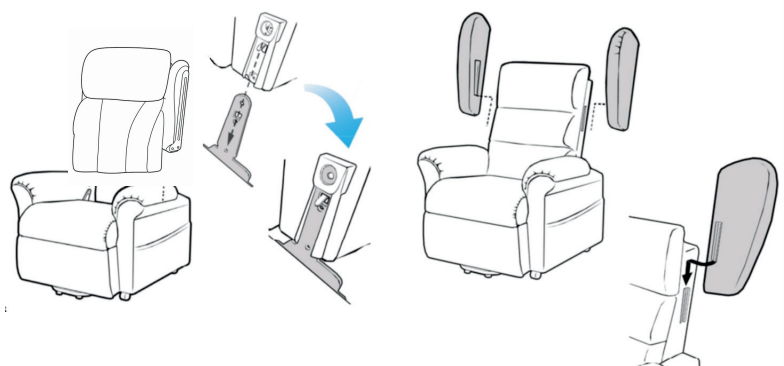

Align the bracket on the right wing with the bracket on the side of the backrest.

To assemble the chair, lower the back brackets onto the vertical flat bars until they are fully inserted on both sides. Note: Both back receivers need to be lowered onto the two flat bars at the same time and evenly.

COMFORT CARE MELBA RECLINER

ELECTRICAL CONNECTIONS

Make sure all the wires are properly connected to ensure proper function.


1. Connector for the motor and wiring
2. Connector for wiring and adaptor
3. Connector for adaptor wiring and plug wiring
4. Remote control connection

COMFORT CARE MELBA RECLINER

Heating Function


Integrated heating pads are built into both the seat and lumbar areas to provide targeted warmth where it matters most. The gentle heat helps promote circulation, reduce muscle tension, and enhance overall comfort during extended sitting.

The heating function is conveniently controlled via intuitive buttons located on the armrests, allowing users to adjust warmth easily without the need to reach for a remote. This accessible control design ensures simple, user friendly operation for individuals with limited mobility or dexterity.

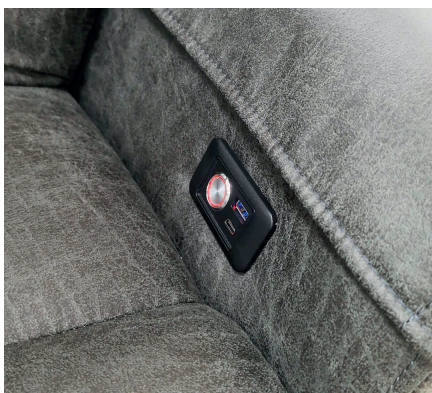

COMFORT CARE MELBA RECLINER

HANDSET CONTROL FUNCTIONS

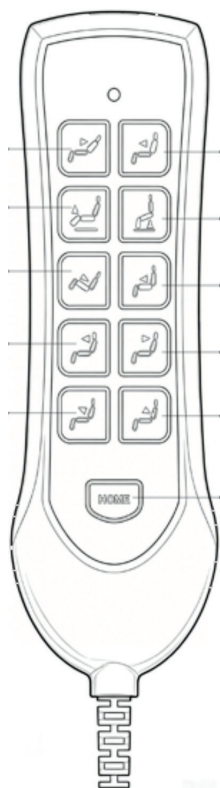

1. Returns Backrest
2. Lowers Backrest
3. Returns Legrest & Lifts Chair
4. Raises Legrest & Lowers Chair
5. Lowers Everything
6. Raises Everything
7. Forwards Headrest
8. Returns Headrest
9. Forwards Lumbar
10. Returns Lumbar
11. Backs to Seating Position

COMFORT CARE MELBA RECLINER

PRODUCT CARE

- Keep the chair away from areas of high humidity and direct sunlight.

Cleaning the chair:

- Before cleaning, ensure that the power is turned off and the plug is removed for the electrical outlet.
- Use a damp cloth and mild soap to clean the plastic, pipe and upholstery. Carefully wipe away all traces of the soap with a dry cloth.
- Do not use harsh chemical cleaning agents.
- Do not remove the upholstery for washing.

Cleaning the remote controls:

Use only dry cloth for cleaning the remote control units. Excess moisture may cause damage.

COMFORT CARE MELBA RECLINER

WARRANTY

Upholstery	The recliner surface leather or fabric is covered by a 5-year warranty, excluding man-made damage.
Okin Motor	The Okin motor is covered by a 2-year warranty, excluding man-made damage.
Warranty Period	The warranty periods above are calculated from the factory release date.
Parts Support	If the covered product parts are damaged within the warranty period, the seller will provide replacement parts free of charge.

COMFORT CARE MELBA RECLINER

MANUFACTURING & IMPORTER INFORMATION

Manufactured by	RECTOR MEDICALGROUPPTE.LTD.
Country of Origin	Made in PRC
Imported by	Ableliving Holding Pty Ltd
Address	65-67 Sheehan Rd, Heidelberg West VIC 3081
Website	https://www.able-living.com/

COMFORT CARE MELBA RECLINER

SPECIFICATIONS

	Standard	Petite
Seat Width	520 mm	480 mm
Seat Depth	520 mm	470 mm
Seat Height	500 mm	460 mm
Total Width	820 mm	760 mm
Total Depth	960 mm	920 mm
Backrest Height	770 mm	750 mm
Armrest Height	180 mm	185 mm
Chair Height (Sitting)	1160 mm	1150 mm
Weight Capacity	155 kg	155 kg
Gross Weight	77 kg	76 kg
Net Weight	72 kg	71 kg
Upholstery (Microsuede)	Manila Toffee	Manila Dark Grey